




WIDEX TV PLAY CONNECTION 1-2-3 QUICK GUIDE

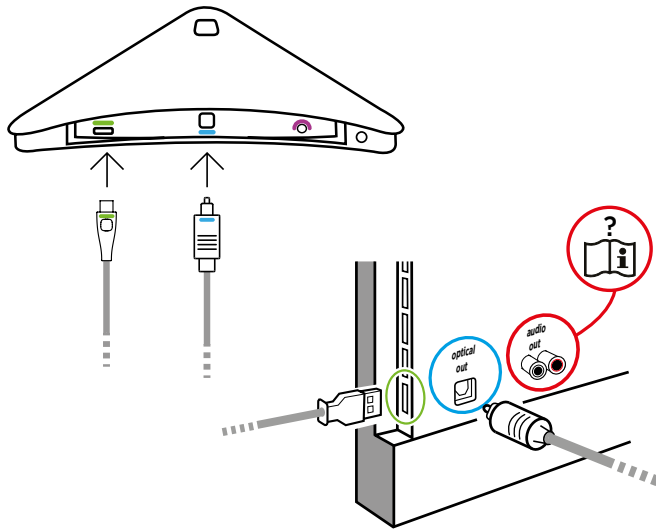

CONNECTION OPTIONS:

Power: Mini USB Cable


- Connect Mini USB side of cable to TV PLAY (white port)
- Connect USB side of cable to TV (can be plugged into the wall if TV doesn't have USB port)

Input: Optical Cord

- Take the plastic caps off the ends of the optical cord before connecting
- Connect the other end of the optical cord to TV


Refer to User Manual for analogue connection.

Refer to User Manual for mounting TV PLAY to the back of the TV.

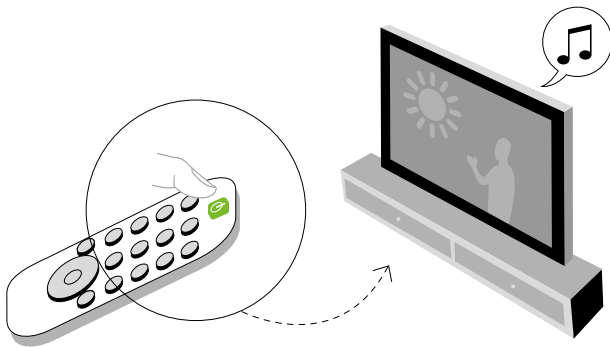

Turn on the TV

Turn on the TV and enjoy TV sound.

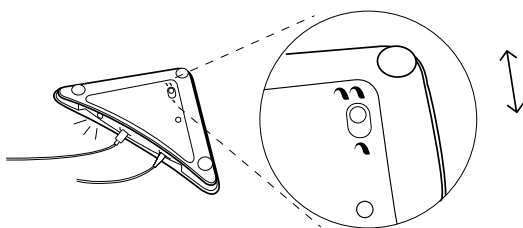


If you only wear one hearing device:

Select the option for one hearing device on the back of the TV PLAY.




WIDEX TV PLAY CONNECTION 1-2-3 QUICK GUIDE

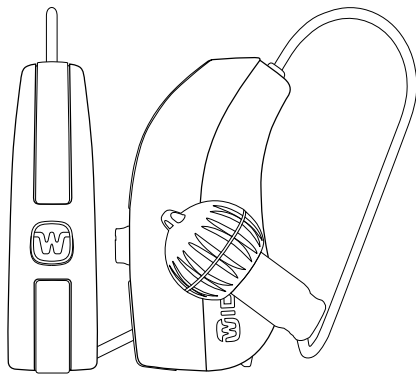

Direct streaming with your hearing aids

Use your Widex EVOKE Fusion2 hearing aids and press the button to select the TV PLAY program.

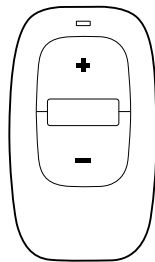

Use your Widex RC-DEX

You can stream directly to Widex TV PLAY by using the button in the middle of the Widex RC-DEX remote control.



Use your Widex EVOKE app

Open the EVOKE app and select the TV PLAY program option.



5 706069 882872

Printed by Rosendahls a/s /2019-06. 9 502 5215 001 #01
Widex, WIDEX EVOKE and DEX are trademarks of Widex A/S.

Widex A/S, Nymoellevj 6, DK-3540 Lyngby WWW.WIDEX.PRO

WIDEX TV PLAY™
STAY CONNECTED